

For Easy, Time saving, User-friendly dental training & education To meet practical demands, ADEM has been evolving day by day.

PER31F-UPLA001 Severe Periodontology Study Model

This model demonstrates the severe periodontology, applying dental culculus on each tooth surface for scaling practice. Soft gingivae facing enables incision and suture for root planing practice.





For practical training of scaling and root planing. Dental culculus is applied to the tooth surface supragingivally and subgingivally.

#26 is extruded due to the loss of #36 while #37 is inclined mesially.

Furcation involvement created on #16,26,36.

Vertical bone resorption on #23











For flap operation, gingivectomy and suture practice





Artificial dental calculus One liquid type. (Optional)

Work with both ADEM Phantom and Simple Manikin





For further inquiry:



TEL: +81-75-256-3015 FAX:+81-75-221-8032 http://www.trimunt.co.jp/